

ECG Monitoring Anytime, Anywhere



KardiaMobile 6L

Getting started with KardiaMobile 6L in 3 easy steps

1

Download the
Kardia app



Available on iOS & Android

2


Record your
ECG in just **30 seconds**

**3**


Reading shared with
your Medanta cardiologist




How to record ECG

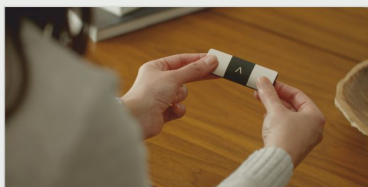

Place your thumbs on the
top electrodes

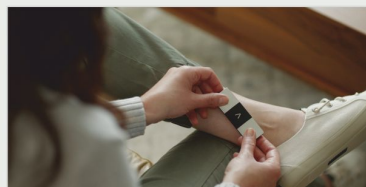

Touch the bottom electrode
to the bare skin of your
left ankle

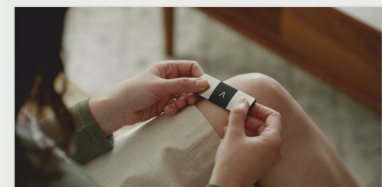

Alternatively, you can place
the bottom electrode to your
left knee

- World's only US FDA-cleared device
 - Hassle-free ECG in 30 seconds
 - No mess of gels and wires
- Detects AFib, Bradycardia, Tachycardia or Normal Sinus Rhythm

More Information:  +91 124 41414 414